



# **Advanced Professional Micro-Needling System**



*Introducing,*  
**An Advanced Professional  
Micro-Needling System**  
*That Is,*

SAFE | EFFECTIVE | TRUSTED



**Quality You Can Trust**

# Professionally Designed for Maximum Comfort, Safety and Best Results



## Multiple Speed Control

Customized Treatment



## Ergonomic Design

Better Grip, Comfortable Handling



## Variable Depth Control

0.5-2.5 mm, Optimum Results



## Secure Pen Stand

Enhanced Device Protection



## EO Sterilized Needle Cartridge

12x33G for Safety, Precision and Accuracy



Scan Here  
to View the Device



## High Tensile Spring

Enhanced Needle Safety and Durability

Skin!n



# Clinical Advantages

- Triggers natural collagen production and skin's own repair mechanism for a better healing and rejuvenation.
- Adjustable speed that provides up to 1300 micro-channels per second for a faster and more efficient treatment.
- Vertical needling helps to ensure cleaner entry and exit without scratching.
- Less epidermal skin damage, minimal pain and discomfort and shorter downtime.
- Better absorption of skin care products applied topically.
- Maintains sterility throughout the procedure for better safety and reduced risk of contamination.
- Can be combined with other therapies like **PRP/PRGF**, chemical peels and drug infusions to further enhance their results.
- **5 Years warranty** when registered on **[www.skinin.org](http://www.skinin.org)**.




Skin!n



# Clinical Applications



Fine Lines and Wrinkles

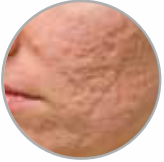

Acne




Scarring



Skin Tightening



Stretch Marks

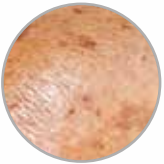

Pigmentation



Enlarged Pores



Hair Loss



# Skin!n Treatment Protocol

## *Step By Step Guide To Perform The Treatment*



Cleanse



Document, Diagnose  
and Before Pictures



Apply Topical  
Anaesthetic



Maintain Sterility



Disinfect  
Treatment Area



Insert Cartridge



Select Needle Depth



Select Speed



Treat the Tx Area



Treatment Direction



Ice Pack and Rest



Other Protocol / Procedures  
and After Pictures

## Healing Phase

### Inflammation

1-5 days

Phagocytosis,  
Growth Factors

### Proliferation


2-21 days

Angiogenesis, Collagen  
Deposition, Fibroplasia,  
GAGS, Re-Epithelialisation,  
Contraction, Apoptosis

### Maturation

20 Days to 24 months

Collagen Replacement,  
Maturation Type III > I

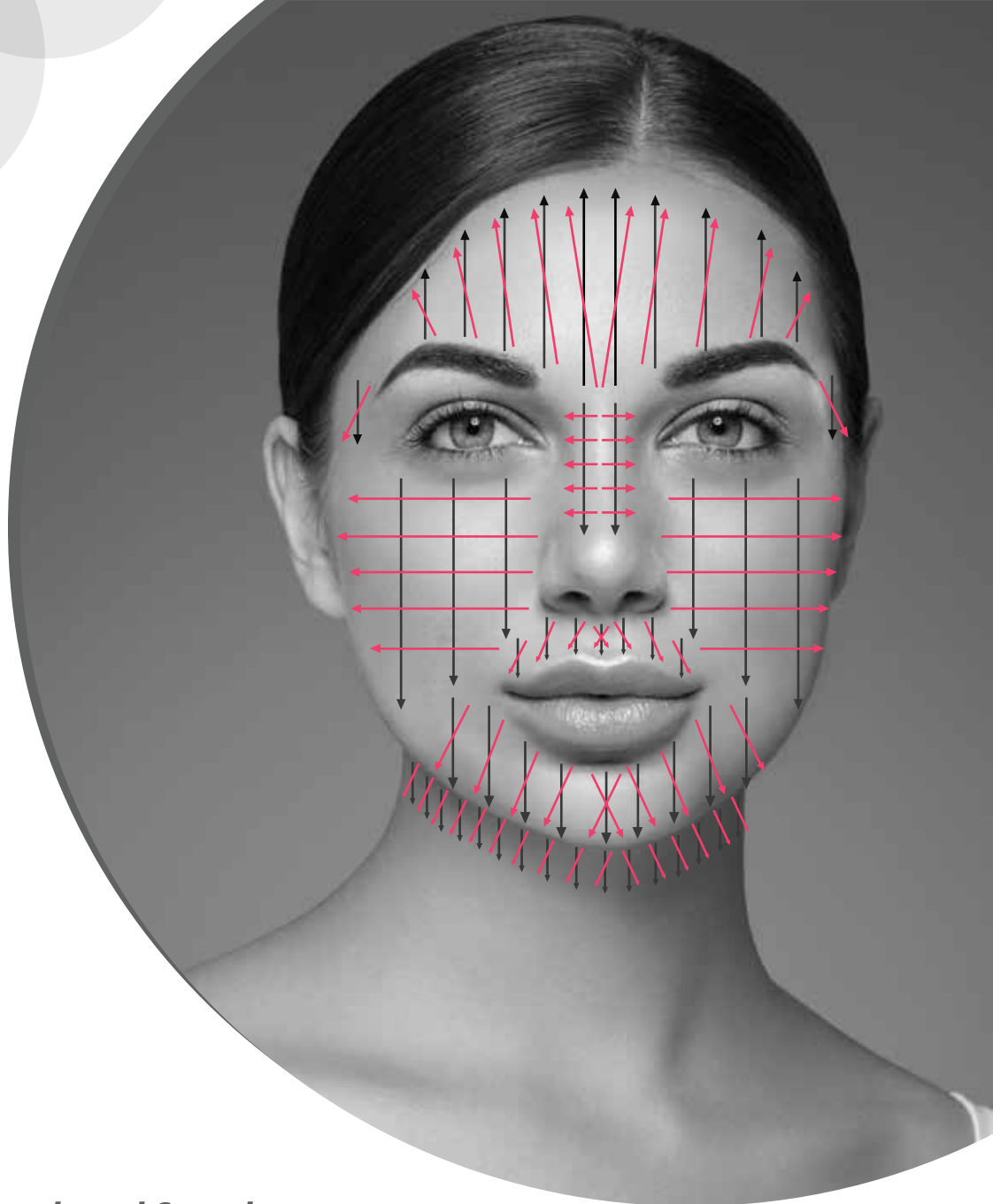

## Skin!n Suggested Depth and Speed

| Region             |   | Needle Depth              |   | Needling Speed         |
|--------------------|---|---------------------------|---|------------------------|
| Forehead           | ⦿ | 0.25-0.5mm                | ⦿ | 4                      |
| Peri-orbital       | ⦿ | 0.25-0.5mm                | ⦿ | 3                      |
| Nose               | ⦿ | 0.25-0.5mm                | ⦿ | 4                      |
| Chin & Jawline     | ⦿ | 0.25-0.5mm                | ⦿ | 4                      |
| Peri-oral          | ⦿ | 0.25-0.5mm                | ⦿ | 4                      |
| Cheeks             | ⦿ | 0.75-2.0mm                | ⦿ | 4                      |
| Back of Arms       | ⦿ | 0.75-1.0mm                | ⦿ | 4                      |
| Back (Acne)        | ⦿ | 0.75-1.5mm                | ⦿ | 4                      |
| Neck & Décolletage | ⦿ | 0.75-1.5mm                | ⦿ | 4                      |
| Breast (Straie)    | ⦿ | 1.5ml-2.0mm               | ⦿ | 4                      |
| Stomach (Straie)   | ⦿ | 1.5ml-2.0mm               | ⦿ | 4                      |
| Thighs & Buttocks  | ⦿ | 0.75-1.5mm                | ⦿ | 4                      |
| Scar Tissue*       | ⦿ | Increase protocol by .5mm | ⦿ | * A general guide only |



**AK Pharma, Inc**  
9100 S Dadeland, 1500-MBP, # 150006  
Miami, FL 33156  
[www.skinin.org](http://www.skinin.org)